
# Chair for showering

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simply relax  
[www.esabya.com](http://www.esabya.com)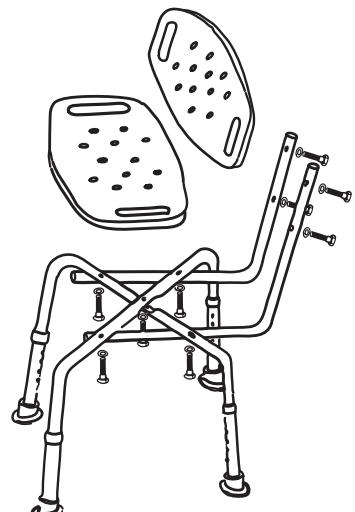

## ENG

### Chair for showering Instruction Manual

A chair complete with backrest, especially designed for use in the bathroom. The chair has built-in drainage holes and handles to improve both hygiene and safety. Extendable legs permit height adjustment while the chair's anti-slip, rubber feet lend excellent stability. Manufactured in aluminium and plastic, totally rust-proof and very light.

- ABS plastic seat in white with an aluminium support frame.
- Chair width: 51 cm.
- Front-to-back length: 36 cm.
- Adjustable height range: from 40 to 50 cm.
- Chair weight: 2.8 Kg.
- Maximum load bearing weight 160 Kg (SGS approved)
- Water-resistant and easy to clean.
- Easy cleaning with a damp cloth

## Assembly

Place the plastic seat upside down on a table. Place the legs in a cross-shape, position the backrest support tubes and secure the assembled parts using the screws provided. After tightening the screws, turn the chair over and attach the backrest to the support tubes.

1. Seat (quantity: 1)
2. Backrest (quantity: 1)
3. Legs (quantity: 2)
4. Support tubes (quantity: 2)
5. Screws (quantity: 9)

## Height adjustment

Press the button located on the legs to adjust height. Extend the leg until the desired height is reached and use the button to lock the leg in position. Repeat the same procedure to adjust each leg. Finally, check that all four feet are adjusted to the same height

## Product cleaning and care

Use a damp cloth, a neutral (mild) soap and dry immediately after washing. Do not immerse the seat in the bathtub, and dry after each use. Do not use corrosive products or cleaning products that contain bleach or similar products, as they may damage or cause discoloration.

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